

Application areas

- High Level Disinfection applications in non-food areas
- Hospitals & healthcare
- Institutional & industrial areas
- Surfaces, tools & equipment
- Travel: boats, buses, planes & trains
- Leisure: sports centres, football stadiums & theme parks



Features & benefits

- Bactericidal
- Fungicidal
- Virucidal
- Fully tested to International Standard EN 14476 against non-enveloped viruses: Adenovirus, Poliovirus & MNV Norovirus
- Fully tested to International Standard EN 14476 against enveloped viruses including Coronavirus
- Tested under dirty conditions against bacteria and fungi

Test results

Tested according to International Standards:

| | Pathogen | Standard | Test Time |
|------------------------------|-------------------------------|----------|-----------|
| Viruses | Adenovirus | EN 14476 | 15 min. |
| | Poliovirus | EN 14476 | 15 min. |
| | Murine Norovirus | EN 14476 | 5 min |
| Bacteria | <i>Enterococcus hirae</i> | EN 1276 | 5 min. |
| | <i>Enterococcus hirae</i> | EN 13727 | 1 min. |
| | <i>Enterococcus hirae</i> | EN 13697 | 5 min. |
| | <i>Escherichia coli</i> | EN 1276 | 5 min. |
| | <i>Escherichia coli</i> | EN 13697 | 5 min. |
| | <i>Listeria monocytogenes</i> | EN 13697 | 5 min. |
| | <i>Pseudomonas aeruginosa</i> | EN 1276 | 5 min. |
| | <i>Pseudomonas aeruginosa</i> | EN 13727 | 1 min. |
| | <i>Pseudomonas aeruginosa</i> | EN 13697 | 5 min. |
| | <i>Salmonella typhimurium</i> | EN 13697 | 5 min. |
| | <i>Staphylococcus aureus</i> | EN 1276 | 5 min. |
| | <i>Staphylococcus aureus</i> | EN 13727 | 1 min |
| <i>Staphylococcus aureus</i> | EN 13697 | 5 min. | |
| Fungi | <i>Aspergillus niger</i> | EN 1650 | 15 min. |
| | <i>Aspergillus niger</i> | EN 13697 | 15 min. |
| | <i>Candida albicans</i> | EN 1650 | 15 min. |
| | <i>Candida albicans</i> | EN 13624 | 5 min. |
| | <i>Candida albicans</i> | EN 13697 | 15 min. |

Application advice

- Available as ready to use liquid or as a concentrate (for dilution at 5% or 1 part to 19 parts water)
- Exhibits excellent cleaning properties - for best results and for High Level Disinfection, pre-cleaning of surfaces is recommended
- Can be applied on surfaces by mop, wipe, lance, spray and surface flooding or by immersion for tools and equipment

Pack sizes

| Concentrate | Case size | Order code | Ready to use (5% dilution) | Case size | Order code |
|--------------------|-------------------|------------|----------------------------|-------------------|------------|
| 5 Litre container | Case of 4 | EV-C-4X5 | 1 Litre trigger sprays | Case of 12 | EV-RTU-1 |
| 20 Litre container | Sold individually | EV-C-20 | 5 Litre container | Case of 4 | EV-RTU-4X5 |
| 200 Litre drum | Sold individually | EV-C-200 | 20 Litre container | Sold individually | EV-RTU-20 |